

REPLACEMENT OF PRESSURE HOSE

1. Elevate chair base to the highest position or as high as the chair base will travel.
2. Remove base covers.
3. Cut all tie wraps holding the black pressure hose.
4. Using a 7/8" wrench loosen the nut on the black hose connected to the motor and pump assembly.



5. Using a 1/2" wrench remove the small nylon from the elbow adapter connected to the pressure hose.



6. Connect the new pressure hose to the motor and pump assembly and to the elbow adapter.
7. Route the hose so that it does not rub against any moving parts or sharp edges.
8. Replace base covers.