

## ADJUSTMENT OF TILT CABLE MANUAL TILT CHAIRS

1. Remove the chair back from the tilt chair.
2. Locate two nuts at the end of the cable near the lock shoes assembly.
3. Using two 7/16 wrenches loosen the silver jam nut while holding the larger black adjustment nut on one of the two cables.
4. Turn the adjustment nut counter clockwise to extend the length of the cable and create excess slack in tilt handle connected to cable being adjusted.
5. Once handle no longer effects the locking of the tilt, make the same adjustment on the other cable.
6. Now choose one of the two cables and turn the adjustment nut clockwise until you remove all but a slight amount of slack in the cable.
7. Compress the adjusted handle and move the chair thru a full range of motion.
8. Using the two 7/16 wrenches hold the adjustment nut while tightening the jam nut on top of the adjusting nut.
9. Repeat steps 3 thru 8 for the other cable.

